

Iron Intern® Hydra Double Arm Retractor SET-UP

HYD022, HYDJ22, HYDJ26




PARTS

Hydra Yoke Double Arm
#HYD0YO


3/D Swinger Rail Clamp
#3DSREG

Hydra Vertical Pole
#POLH22

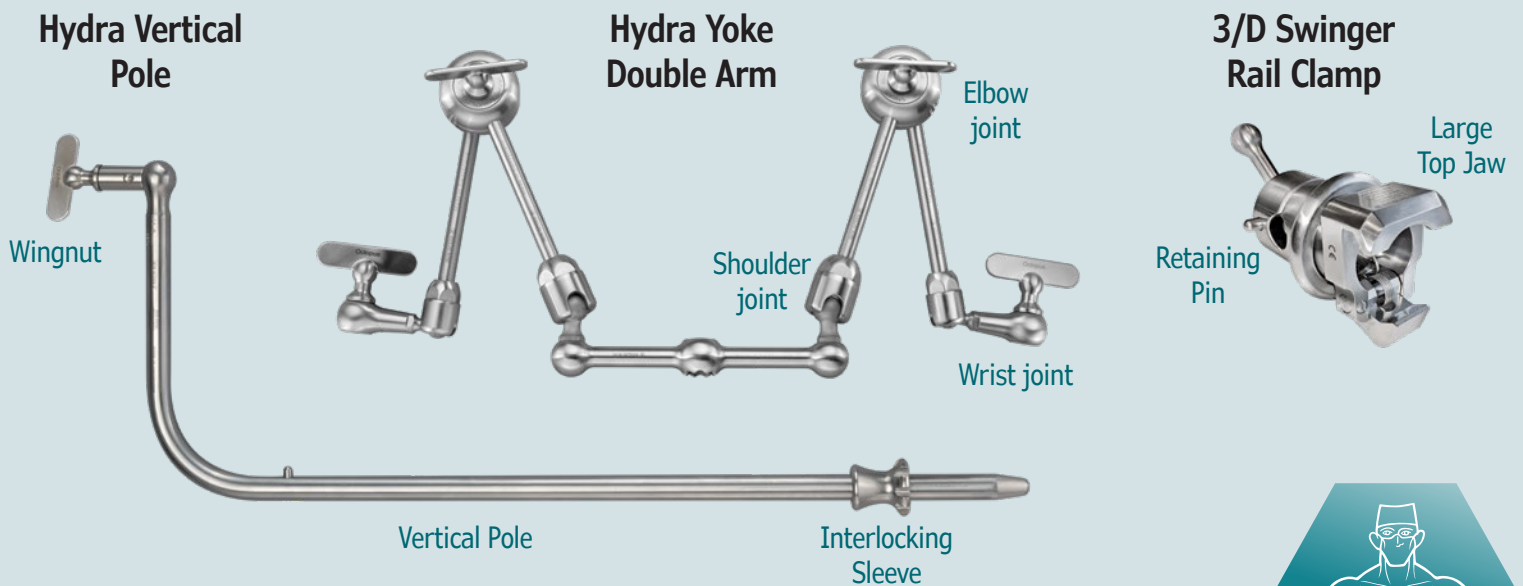

www.IronIntern.com 732-602-7717

View setup instruction videos online: **Iron Intern** on  **YouTube**

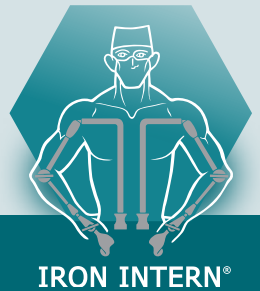

SET-UP INSTRUCTIONS



1 Fully open the 3/D SWINGER RAIL CLAMP by turning the WINGNUT counterclockwise.




2 Attach the 3/D SWINGER RAIL CLAMP onto the bed rail by hooking the larger and wider jaw onto the bed rail edge with the lower jaw resting underneath the bed rail.



3 Insert the VERTICAL POLE through the top of the open 3/D SWINGER RAIL CLAMP. Swivel the POLE to turn the shoulder to the desired position.



4 Move the INTERLOCKING SLEEVE to sit on top of the 3/D SWINGER RAIL CLAMP. Align the sleeve notch with the RETAINING PIN to secure.

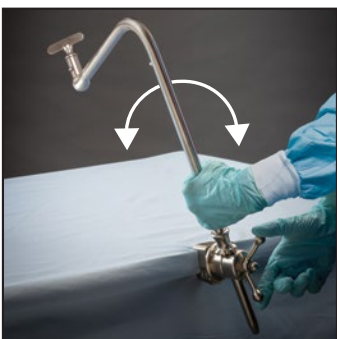


5 Adjust the VERTICAL POLE to the required vertical, horizontal or diagonal position, not to exceed 180° rotation. Secure the 3/D SWINGER RAIL CLAMP by tightening the WINGNUT clockwise.




6 Unscrew the WINGNUT on the VERTICAL POLE and attach the HYDRA YOKE making sure **THE SCALLOPED EDGE OF THE YOKE FACES DOWN.**



7 Screw the WINGNUT back into place and tighten to secure the Iron Intern® HYDRA YOKE.

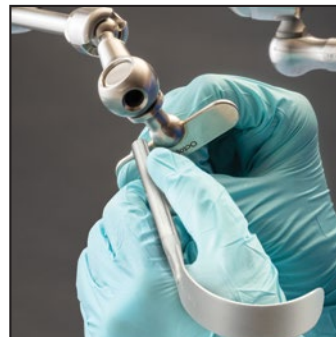

8 Insert a BLADE HANDLE into the WRIST JOINT and secure by turning the WRIST WINGNUT clockwise. Repeat on the opposite arm.

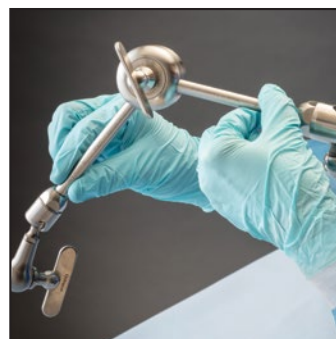

9 Position the HYDRA ARMS as desired.



10 Secure the HYDRA ARMS by tightening the ELBOW WINGNUTS. Loosen the ELBOW WINGNUTS to allow manipulation of the HYDRA ARMS.