

# Iron Intern® Single Arm Retractor SET-UP

ARH017, ARH022, ARHD22, ARHJ22,  
ARH026, ARHJ26



## PARTS


Iron Intern® Single Arm  
#ARH022-ARM

3/D Swinger Rail Clamp  
#3DSREG

### 3/D Swinger Rail Clamp



### Iron Intern® Single Arm

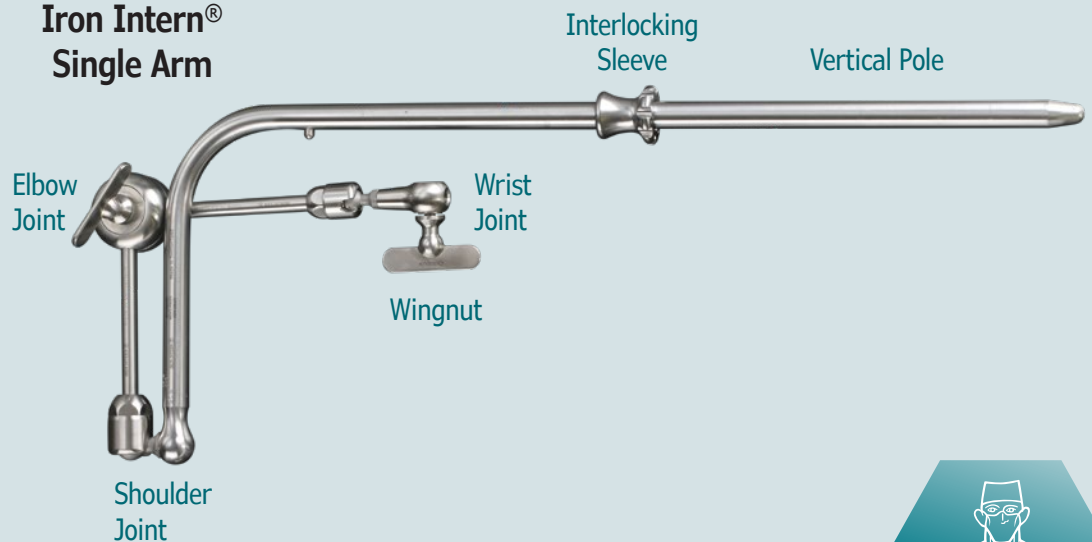

[www.IronIntern.com](http://www.IronIntern.com) 732-602-7717

View setup instruction videos online: **Iron Intern** on  **YouTube**



**IRON INTERN®**

## SET-UP INSTRUCTIONS




**1** Fully open the 3/D SWINGER RAIL CLAMP by turning the WINGNUT counterclockwise.




**2** Attach the 3/D SWINGER RAIL CLAMP onto the bed rail by hooking the larger and wider jaw onto the bed rail edge with the lower jaw resting underneath the bed rail.




**3** Insert the VERTICAL POLE through the top of the open 3/D SWINGER RAIL CLAMP. Swivel the POLE to turn the shoulder joint to the desired position.

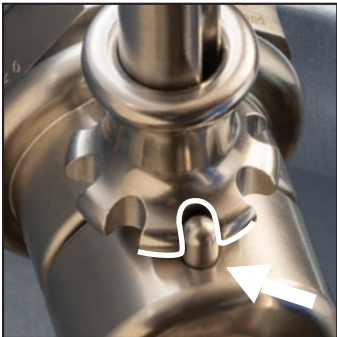

**4** Move the INTERLOCKING SLEEVE to sit on top of the 3/D SWINGER RAIL CLAMP. Align the sleeve notch with the RETAINING PIN to secure.

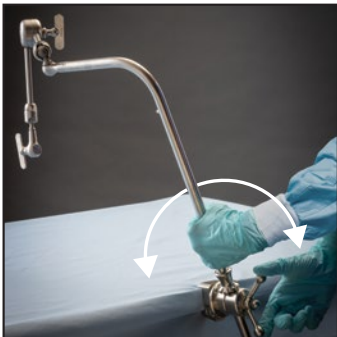

**5** Adjust the VERTICAL POLE to the required vertical, horizontal or diagonal position, not to exceed 180° rotation. Secure the 3/D SWINGER RAIL CLAMP by tightening the WINGNUT clockwise.

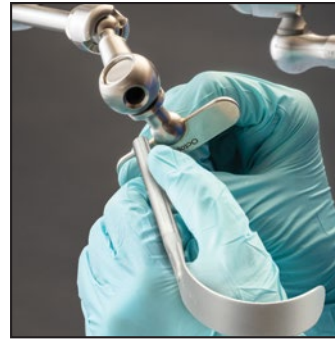

**6** Insert a BLADE HANDLE into the WRIST JOINT and secure by turning the WRIST WINGNUT clockwise.



**7** Position the IRON INTERN® SINGLE ARM as desired.



**8** Secure the Iron Intern® SINGLE ARM by tightening the ELBOW WINGNUT. Loosen the ELBOW WINGNUT to allow manipulation of the Iron Intern® SINGLE ARM.